


QUICK GUIDE - HOW TO USE THE INTEREAR FEATURE

This quick guide describes how to use the InterEar feature. The InterEar feature is found under Finalise – HA configuration. The feature is only active if you have a binaural fitting.

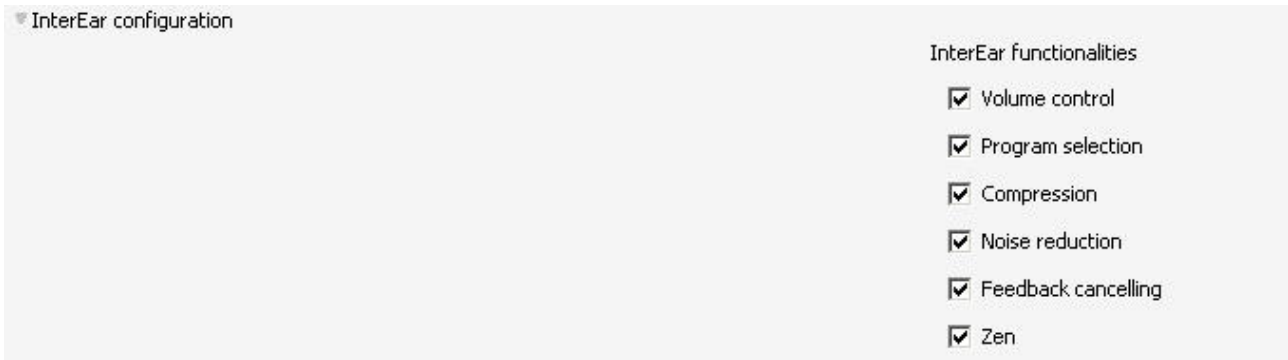

Depending on the fitting there are different options regarding InterEar functionalities. The default set of functionalities are:

- Volume control
- Program selection
- Compression
- Noise reduction
- Feedback cancelling
- Zen

The InterEar functionalities are active by default if you have a binaural fitting.

The functionalities are deactivated by unmarking the checkbox.

If the hearing aid is matched to a DEX, the two items Volume control and Program selection cannot be turned off. In this case these items are greyed out.

If you are working with only one hearing aid, this panel contains a text informing you that the InterEar functionalities do not work for monaural fittings.