

QUICK GUIDE - HOW TO USE FEATURE SETTINGS

This quick guide describes how to use Feature settings. The Feature settings panel is found under Fine tuning.



The features that can be changed are:


- Microphone modes
- Speech and noise modes
- Impulse sound modes
- Feedback cancelling modes

All listening programs have their individual set of features according to the purpose of the program. The features can be changed individually for each program.

If the right and left hearing aid are linked together, a change in features settings in one of the hearing aid will also influence the other hearing aid.

The hearing aids can be unlinked by clicking this symbol 

The unlink symbol is found between the right and left hearing aid.

 indicates that the feature has an InterEar functionality. The InterEar functionality can be disabled under Finalise.