


## QUICK GUIDE


# - HOW TO USE THE GAIN MATRIX

This quick guide describes how to use the Gain matrix. The Gain matrix is found under Fine tuning.

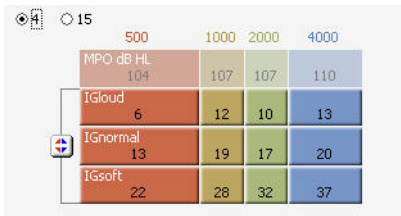

The Gain matrix opens up in four frequency regions for soft, medium and loud inputs.

The areas can be adjusted one by one or two or more together. To adjust one area, click on the area that you want to adjust. The selected area opens up with the possibility to increase or decrease gain.

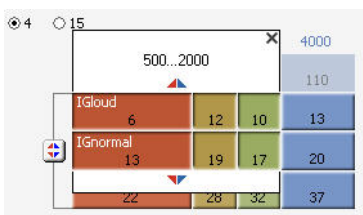

You can select several parameter fields and change their settings simultaneously by placing the cursor on the first parameter, clicking the mouse button and dragging it across all the parameters you wish to adjust.



If you need to do more frequency specific gain adjustment, the Gain matrix can be opened in an extended version. Click the radio button for 15 channels above the Gain matrix to get access to the Extended Gain matrix.

To adjust for all input levels and channels use the “select all” function next to the panel.

Target gain for the specific channel can be viewed using the Target icon. Targets are displayed as targets for the specific program.



The Recalculate tool recalculates the setting in the current fitting. This overrules any adjustments you have made and sets the hearing aid back into the setting that Compass originally recommended.

