


## QUICK GUIDE

# - HOW TO USE THE ACCLIMATISATION TOOL

This quick guide describes how to use the Acclimatisation tool. The Acclimatisation tool is found under Fine tuning, in the Master program.

First-time hearing aid users may need a special setting with reduced gain in the hearing aids to get used to the hearing aids. Compass offers a special acclimatisation tool in Fine tuning for first-time users. The tool is active in the Master program, but will influence all programs that are linked to the Master program.

The Acclimatisation tool has four steps; step 1 is the softest, and 4 is equal to the default gain calculation in the Master program in CLEAR440 based on the results of the Sensogram and Feedback test. Step 2 is equal to the Acclimatisation program available for ISP and Dual ISP hearing aids.

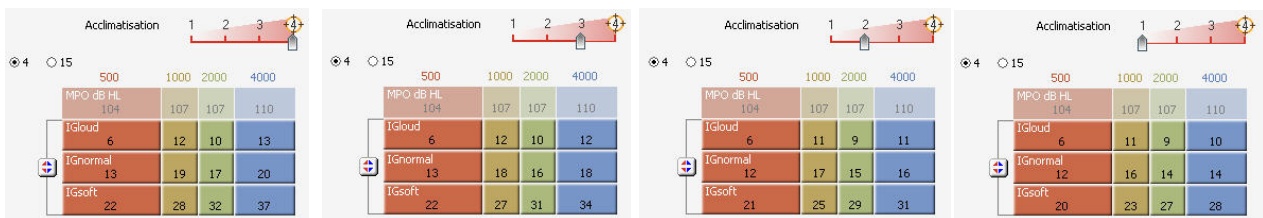

Move the slider to the wanted Acclimatisation step. The acclimatisation adjustment is now active in all programs in the hearing aid that are linked to the Master program.

After an acclimatisation period, Acclimatisation should be moved up to the default gain setting, step 4. The Acclimatisation tool is not an automatic tool, the steps should be manually adjusted when the user is coming back for fine tuning and follow-up.