WIDEX A/S

Nymoellevej 6, DK-3540 Lyngby, Denmark
<https://global.widex.com>

Manual no.:
9 514 0380 081 #03
Issue:
2020-01


DEX™

**COM-DEX
Remote Mic**


GB
NL
PT
NO
SE
FI
LT
RU
LV
PL

WIDEX®
HIGH DEFINITION HEARING


1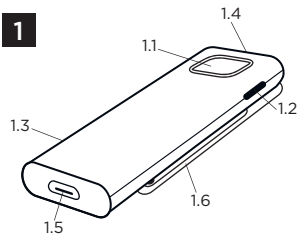


2

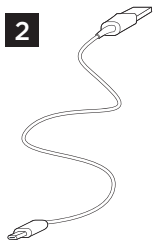
3

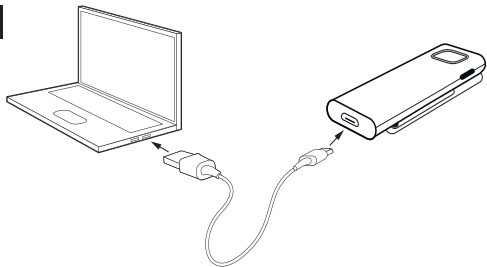4

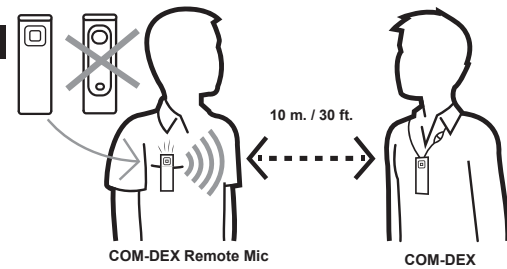5
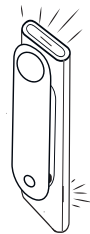
| | | |
|----|----------------------------------|-----|
| GB | USER INSTRUCTIONS | 4 |
| NL | GEBRUIKSAANWIJZING | 14 |
| PT | INSTRUÇÕES DE UTILIZAÇÃO | 24 |
| NO | BRUKSANVISNING | 34 |
| SE | BRUKSANVISNING | 44 |
| FI | KÄYTTÖOHJE | 54 |
| LT | NAUDOJIMO INSTRUKCIJOS | 64 |
| RU | ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ | 74 |
| LV | LIETOTĀJA INSTRUKCIJA | 85 |
| PL | INSTRUKCJA OBSŁUGI | 95 |
| | FCC AND ISED STATEMENTS | 106 |

DESCRIPTION OF DEVICE - see illustration 1

1.1 Streaming button

1.2 Power button

1.3 LED

1.4 Microphone and LED

1.5 Charging port

1.6 Clip

Intended use

The COM-DEX Remote Mic device is a microphone with an integrated radio intended for transmitting sound to a COM-DEX connected to the user's hearing aid(s). The aim is to improve the hearing aid user's ability to hear speech in difficult listening situations. **For optimal connectivity the device and the COM-DEX must be within sight of each other.**

NOTE

Your Remote Mic is only intended to be used with a COM-DEX (with firmware V1.3 or later).

Accessories - see illustration 2

USB cable

USING YOUR DEVICE

Charging your device - see illustration 3

Connect the USB cable to your device and the power supply. It takes approximately 3 hours to charge your device.

NOTE

Before you can use the device, you must disconnect the USB cable.

Battery life: 8 hours streaming.

Turning the device on and off

To turn on your device:

- Press the power button and keep it pressed for 3 seconds.

To turn off your device:

- Press the power button and keep it pressed for 3 seconds.

The device turns off automatically if it has not been streaming sound for more than 30 minutes.

NOTE

To achieve the best sound possible, always turn your COM-DEX Remote Mic off when not in use.

Pairing the device

Pairing your device to your COM-DEX

To pair your device to your COM-DEX for the first time, follow these steps:

1. Make sure that your COM-DEX is in pairing mode.
2. Turn on your device. The first time you do this, the device will be set in pairing mode and it will automatically connect to your COM-DEX. This can take a few minutes.

NOTE

Once you have paired your device to your COM-DEX, it will remain so and will connect automatically if your COM-DEX is on.

Resetting the COM-DEX Remote Mic for a new pairing:

1. Turn on your device.
2. Reset your device to factory settings by pushing both buttons for 5 seconds. The device will be set to pairing mode.
3. Follow the pairing procedure described above.

Using the device - see illustration 4

Remove the protective film before using your device.

Place the device on the shirt collar of the person you'd like to hear.

Turn on the device. The Remote Mic starts streaming automatically.

NOTE

While streaming, your hearing aid microphones will be turned off. Once the Remote Mic stops streaming, the hearing aid microphones will be turned on again after a few seconds. If you want to turn on your hearing aid microphones while streaming, simply press the Room Off button on your COM-DEX.

To start and stop streaming, simply push the streaming button on your Remote Mic. While streaming, the LED will show a green light.

NOTE

If you want to operate your device via the COM-DEX app, you'll need to update the app.

Sound adjustment

The sound that is streamed from the device to COM-DEX can be adjusted in 3 levels. Simply push the power button on the side of the device and the sound level will be altered in a three step circle (highest level – lowest level – back to normal level).

Visual indicators - see illustration 5

Your device has two LEDs that flash with different colours. The purpose of these colours is to inform you of which mode your device is in at any given time.

- Constant red light on the **bottom** LED: The device is charging.
- Constant green light on the **bottom** LED: The device is fully charged.
- Flashing blue-red-blue-red light on **both** LEDs: The device is ready to be paired.

- Flashing blue light on the **top** LED: The device is on, but there is no connection to the paired COM-DEX.
- Flashing green light on the **top** LED: The device is on and connected to the COM-DEX, but it is not streaming.
- Constant green light on the **top** LED: The device is on and streaming.
- Occasionally flashing red light on **both** LEDs: The battery is low.

Maintenance

- Clean the device with a damp cloth. Do not use any liquids or chemicals and do not immerse in liquids.
- When the device is not in use keep it in a dry location out of reach of children and pets.
- Never try to open or repair the device yourself. This may only be done by authorised technicians.

TROUBLESHOOTING

| Problem | Possible cause | Solution |
|--------------------------|--|---|
| The device does not work | 1. Remote Mic battery low | 1. Charge Remote Mic battery |
| | 2. COM-DEX battery low | 2. Charge your COM-DEX |
| | 3. Out of range | 3. Keep the devices in sight of each other and in range (up to 10m) |
| No sound | 1. Not paired to COM-DEX | 1. Repeat the pairing process |
| | 2. COM-DEX is out of the hearing aids' range | 2. Move into range |
| | 3. Hearing aid batteries are low | 3. Change the hearing aids' batteries |

If the problem persists, contact your hearing care professional. For more information, visit: <https://global.widex.com>.

Warnings

Do not leave the device unattended while charging.

Your device should be stored and transported within the temperature and humidity ranges of -10° to +50°C (14°F to 122°F) and <90% RH.

Your device is designed to operate from 5°C (41°F) to 40°C (122°F).

Keep the device protected from dust or external particles.

We recommend using a 5V up to 500 mA USB adaptor.

Do not use the device in mines or other areas with explosive gases.

When the device is connected via micro USB cable to external mains-operated equipment such as a charger, PC, audio source or similar, this equipment must comply with IEC 60950-1, IEC 62368-1, IEC 60601-1 or equivalent safety standards.

Only the accessories and cable that are included in the sales box must be used. Using cables other than those specified or provided may result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

REGULATORY INFORMATION

Directive 2014/53/EU

Widex A/S hereby declares that this COM-DEX Remote Mic is in compliance with the essential requirements and other relevant provisions of Directive 2014/53/EU.

The device contains radio transmitters operating at: 2.4 GHz, 0.9 mW EIRP.

A copy of the Declaration of Conformity can be found at:

<https://global.widex.com/doc>

N26346

Information regarding disposal

Do not dispose of hearing aids, hearing aid accessories and batteries along with ordinary household waste.

Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for electrical waste and electronic equipment, or given to your hearing care professional for safe disposal.

SYMBOLS

Symbols commonly used by Widex A/S in medical device labelling (labels/IFU/ etc.)

| Symbol | Title/Description |
|--------|-------------------|
|--------|-------------------|

**Manufacturer**

The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.

**Catalogue number**

The product's catalogue (item) number.

**Consult instructions for use**

The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.

**Warning**

Text marked with a warning symbol must be read before using the product.

**WEEE mark**

“Not for general waste”. When a product is to be discarded, it must be sent to a designated collection point for recycling and recovering to prevent the risk of harm to the environment or human health as a result of the presence of hazardous substances.

| Symbol | Title/Description |
|---------------|--------------------------|
|---------------|--------------------------|

| |
|----------------|
| CE mark |
|----------------|

| |
|---|
| The product is in conformity with the requirements set out in European CE marking directives. |
|---|

| |
|-----------------|
| RCM mark |
|-----------------|

| |
|---|
| The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand markets. |
|---|

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FCC ID: TTY-CDRM

IC: 5676B-CDRM

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by could void the user's authority to operate the equipment.

ISED Statement / Déclaration d'ISED

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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ISED RADIATION EXPOSURE STATEMENT:

This equipment complies with ISED RF radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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